Symbols Description / Opis symbol / Opis symbol / Szimbólumok leírása / Opis symbol / Opis symbol / Opis simbola

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The **Bluetooth** word and logos are registered trademarks owned by **Bluetooth SIG, Inc.** and any use of such marks by OMRON HEALTHCARE Co., Ltd. is under license. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries and regions. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. Other trademarks and trade names are those of their respective owners.

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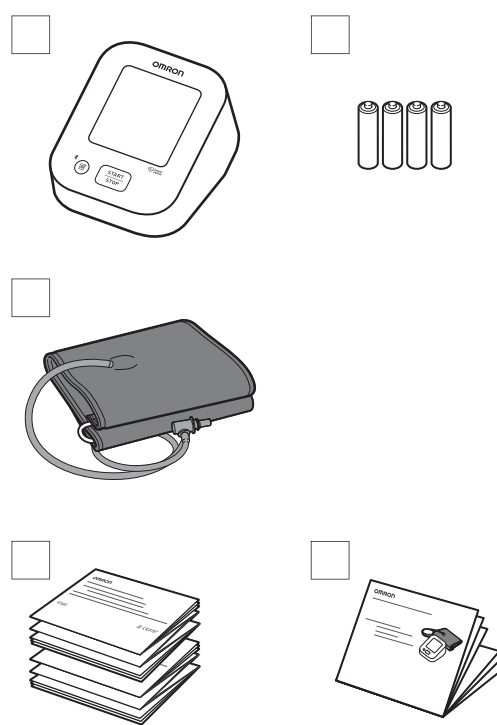
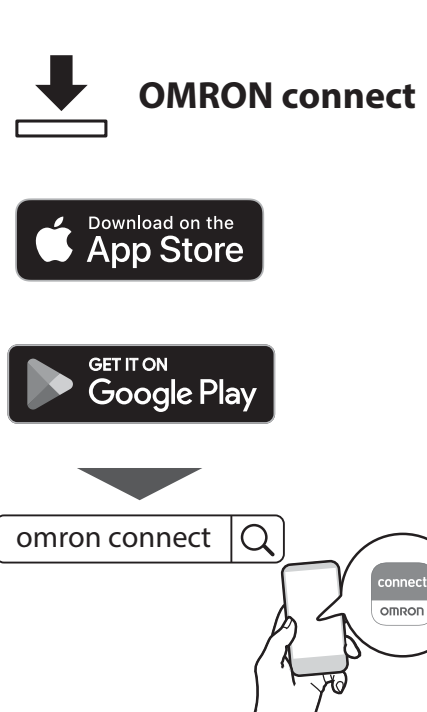
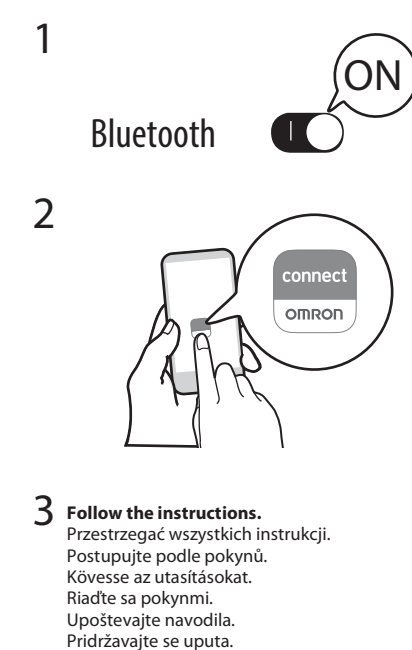
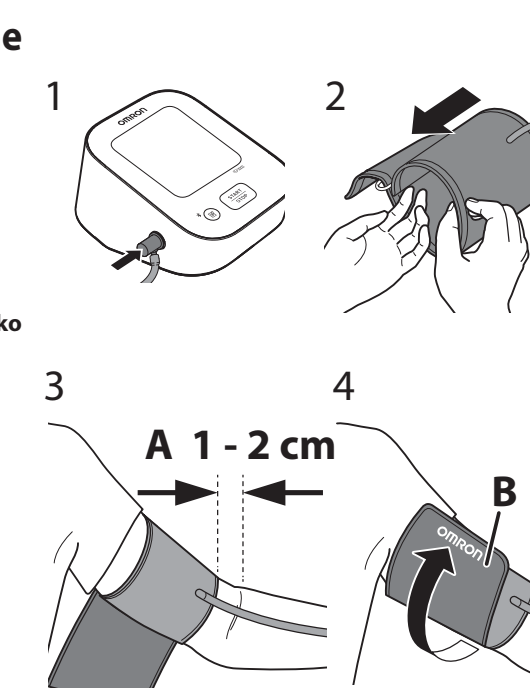
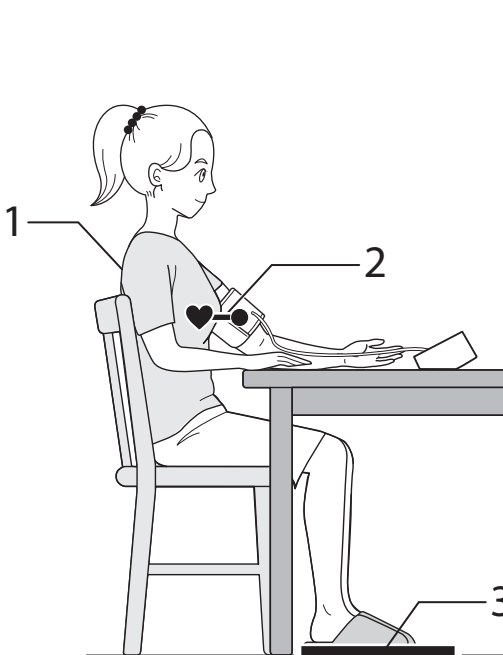
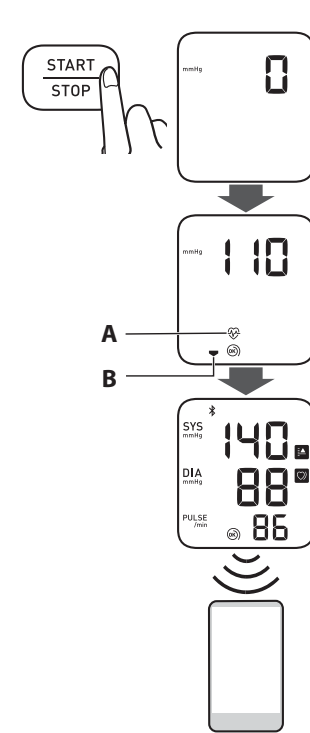
Instruction Manual 2: Operational Instructions

- PL Instrukcja obsługi 2: Instrukcje użytkowania
- CZ Návod k obsluze 2: Provozní pokyny
- HU 2. Használati útmutató: Kezelési utasítás
- SK Návod na použitie 2: prevádzkové pokyny
- SL Priročnik z navodili 2: Navodila za upravljanje
- HR Priručnik s uputama 2: Upute za rad

Read Instruction Manual 1 and 2 before use.

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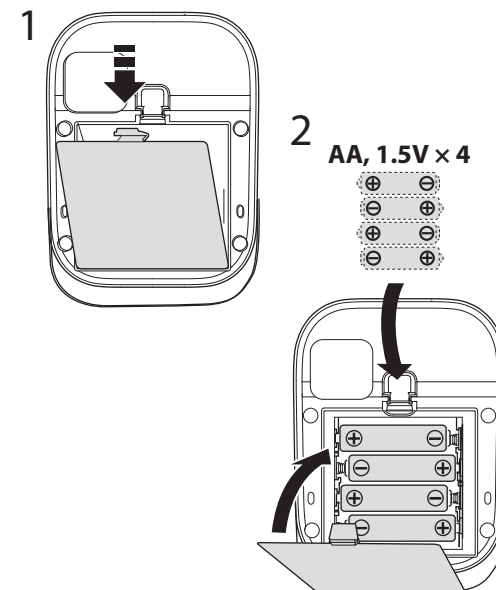
- A. Tube side of the cuff should be 1 - 2 cm above the inside of your elbow.
- B. Make sure that the air tube is on the inside of your arm and wrap the cuff securely so it can no longer slip round.

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- Relax and sit comfortably. Remain still and do not talk.
- 1. Keep your back and arm supported.
- 2. Keep the arm cuff at the same level as your heart.
- 3. Keep your feet flat and your legs uncrossed.

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- 1. Press the [START/STOP] button.
- A: Flashes at every heartbeat.
- B: Appears while the cuff is deflating.
- 2. The reading is saved automatically.
- Open the app to transfer the reading.

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